



LCR Hallcrest Ltd.  
Riverside Buildings, Dock Road,  
Connah's Quay, Flintshire,  
CH5 4DS U.K.  
Tel: +44 (0)1244 817107  
Fax: +44 (0)1244 818502  
Email: sales@lcrhallcrest.com  
Web: www.lcrhallcrest.com

---

Advanced Thermographic Technologies

## Brite Event®

---

### Product Data Sheet

---

**Product Description: Brite Event Urine Temperature Indicators**

---

**Standard Range: 32°C-39°C / 90-100°F**

---

#### Construction:

Adhesive Tape	Double-coated Adhesive Tape
Covering Film	Polyester 3 mil stock
Colour Change Material	Non toxic mixture of liquid crystal esters
Accuracy	±0.5°C / ±1°F
Shelf Life	12 months from invoice date when stored at room temperature (i.e. 21°C (70°F) & 50% relative humidity).

---

**Application Data:** Apply temperature indicating label to outside of urine specimen cup.  
Read temperature after colours stop changing. Green is correct reading. If green is not visible, subtract 1° from highest lit event.

**Function:** Display of temperature, as indicated by green event, of urine specimen cup. Label is adjusted with a 1°C offset.

**Use:** Ideal for use in clinics, hospitals, and doctors' offices.