

## Clinitrend<sup>®</sup>

---

### Instructions for Use

---

**Product Description:** L320-100 Latex Free Forehead Moving Line Temperature Monitor

---

**Standard Range:** 29-41°C / 84-106°F

**Indication:** Monitor trends and / or measure human forehead surface temperature

**Use:** Ideal for use in Pre-op, Surgery, PACU or wherever temperature trend monitoring is indicated.

Forehead Thermometer readings are different from and will not give the same reading as an oral or rectal thermometer. Forehead temperature is estimated to be 1.94°C/3.5°F lower than core body temperature.

Clinitrend has an offset of 1.94°C/3.5°F that compensates for this difference by increasing the displayed temperature by 1.94°C/3.5°F. This offset can differ person to person and therefore the temperature displayed on the strip could vary from actual core temperature.

Unusual readings should be verified with an oral or rectal thermometer

**Directions:** Remove backing. Apply to centre of forehead. Skin surface should be clean and dry.

Read temperature when yellow line stops moving.

Reversible analogue line moves along the scale to show temperature readings in °C and °F.

**Operating conditions:** 15 to 40°C / 60 to 104°F

**Storage:** 5 to 65°C / 41 to 149°F