

Brite Event®

Product Data Sheet

Product Description: Brite Event Urine Temperature Indicators

Standard Range: 32°C-39°C / 90-100°F

Construction:

Adhesive Tape	Double-coated Adhesive Tape
Covering Film	Polyester 3 mil stock
Colour Change Material	Non toxic mixture of liquid crystal esters
Accuracy	±0.6°C / ±1°F
Shelf Life	12 months from invoice date when stored at room temperature (i.e. 21°C (70°F) & 50% relative humidity).

Application Data: Apply temperature indicating label to outside of urine specimen cup.
Read temperature after colours stop changing. Green is correct reading. If green is not visible, subtract 1° from highest lit event.

Function: Display of temperature, as indicated by green event, of urine specimen cup. Label is adjusted with a 1°C offset.

Use: Ideal for use in clinics, hospitals, and doctors' offices.